ENGLISH DENTAL ADHESIVE MATERIAL FOR CERAMICS

PRECAUTIONS FOR DENTAL PERSONNEL AND PATIENTS

- Do not use this material on patients with known allergies to resin based chemicals (methacrylate monomer or acetone).
- If any inflammation or other allergic reactions occur on patient or operator when using this product, immediately discontinue use and seek medical advice.
- Avoid contacting soft tissues, skin or eyes. In case of contact, rinse immediately with copious amount of water and if necessary, seek medical advice.
- This product is intended only for dental professional use.
- Use in well-ventilated areas to avoid inhalation of acetone.

APPLICATION

Adhesion for dental ceramic restorations or devices

DIRECTIONS FOR USE

- Pre-treatment: for seating Alumina or Zirconia dental ceramic restorations using dental adhesive resin cement
- Pre-treatment: for repairing Alumina or Zirconia dental ceramic restorations using dental filling composite resin

- Pre-treatment for adhesive surface
 - Intraoral use**
 - Roughen adhesive surface using dental abrasives (diamond point etc). Rinse with water and dry.
 - Extraoral use**
 - Sandblast with 50-100 µm alumina at gauge pressure of about 0.2-0.3 MPa (about 2-3 kgf/cm²). Rinse with water and dry.
- Application of AZ-Primer
 - Dispense adequate amount of AZ-Primer on a cish etc. Apply a uniform layer on the entire adhesive surface using a microbrush etc and leave it undisturbed for 10 seconds.
- Seating / repairing of dental crown restorations
 - Follow the directors of adhesive resin cement manufacturer.

NOTE ON USE

- Since this product includes acetone, tightly close the cap immediately after each dispensing and dispensed material must be used promptly.
- If treated restoration surface gets contaminated with saliva or blood etc., perform ultrasonic cleaning with soap and rinse thoroughly under running water.
- In case where the contamination with saliva or blood etc occurs after the application of this product, clean with an alcohol moistened cotton pellet: and reapply the material.
- Store in dark place at room temperature (1-30 °C) keeping away from any source of ignitor: and direct sunlight
- Do not mix this product with any other products.
- To avoid cross-infection, dispensec primer should be used on only one patient.

COMPOSITION

AZ-Primer: Acetone, Phosphonic acid monomer, etc

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